



Due to the increased hygiene requirements and extensive mask obligation we have developed the SynFlex FaceShield with materials from our existing product portfolio. It is manufactured with the help of our laser system at the Schramberg location.

- light and high-quality face visor
- anti-fog coating
- high wear comfort
- safe fixing
- visible facial expression
- easy fitting
- suitable for spectacle wearers
- suitable for the additional use of breathing masks
- easy cleaning

## Prices

For every FaceShield sold, we give one face visor **free of charge** to a charitable institution.

| Quantity     | Price per piece |
|--------------|-----------------|
| from 10 pcs  | 7.90 €          |
| from 50 pcs  | 6.50 €          |
| from 100 pcs | 5.80 €          |

No shipping costs.

All prices are net plus value added tax.

Content per packaging unit 10 pieces.

## Assembly

You can access our assembly video via our homepage [www.synflex.de/news/faceshield](http://www.synflex.de/news/faceshield) or via our YouTube channel: **SynFlex Group – Resource of Power**.

## Order

Please contact your customer service as usual to place FaceShields orders or e-mail us at [faceshield@synflex.de](mailto:faceshield@synflex.de).

You are a non-profit organisation and are interested? Please e-mail us at [faceshield@synflex.de](mailto:faceshield@synflex.de).


## Notes

The product serves as protection against splashes and droplets when in contact with persons in the immediate vicinity. It is no substitute for mouth-nose protection. The product has not been tested according to any standard and is not a medical product.

Open packings are non-returnable for hygienic reasons.

Our offers are addressed exclusively to companies and tradesmen.

## Assembly

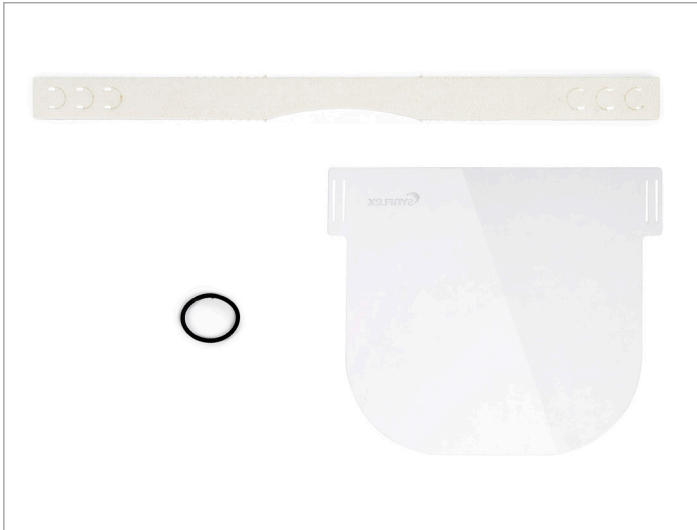

### Step 1:

Place the visor and bandeau in front of you as shown. Make sure that the SynFlex logo is in the upper left corner of the visor and that the fleece side of the bandeau is facing you. The curve in the middle points downwards.



### Step 2:

Thread the ends of the bandeau sideways through the slits in the visor.

Use the locking points on the bandeau to adjust and fix the distance from the visor to the face.



### Step 3:

Thread the supplied rubber through the eyelets of the bandeau. The different eyelets allow a multi-stage adjustment to the head circumference.

For small head circumferences, the bandeau can be shortened with scissors to cut off unused eyelets.