


Assembly

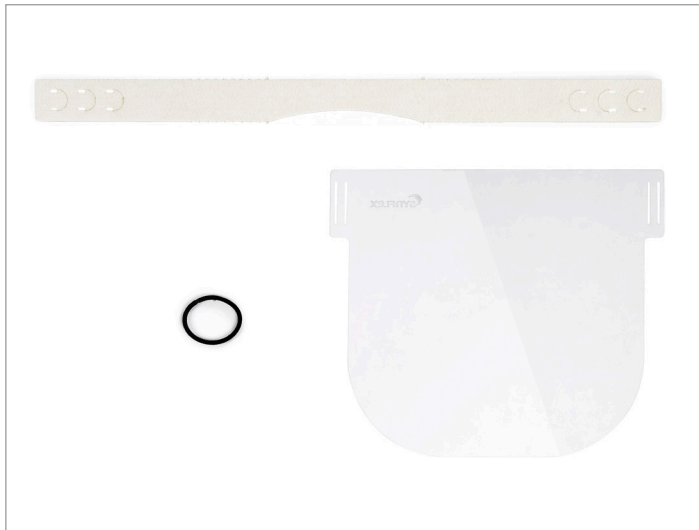

Step 1:

Place the visor and bandeau in front of you as shown. Make sure that the SynFlex logo is in the upper left corner of the visor and that the fleece side of the bandeau is facing you. The curve in the middle points downwards.



Step 2:

Thread the ends of the bandeau sideways through the slits in the visor.

Use the locking points on the bandeau to adjust and fix the distance from the visor to the face.



Step 3:

Thread the supplied rubber through the eyelets of the bandeau. The different eyelets allow a multi-stage adjustment to the head circumference.

For small head circumferences, the bandeau can be shortened with scissors to cut off unused eyelets.